


Notice de démontage/remontage pour nettoyage Positionneur de guide de coupe huméral

Références concernées	Désignation	Nombre de composants
B2293311	Positionneur de guide de coupe huméral	3

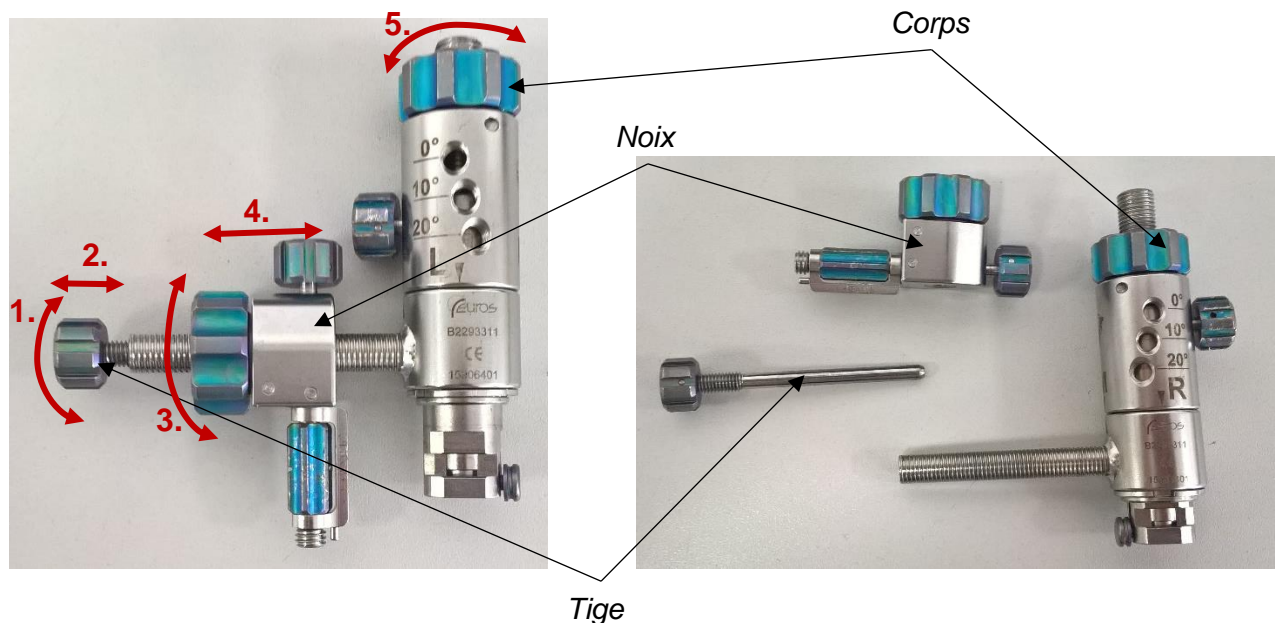

Etapes de démontage/remontage pour nettoyage


Pour le nettoyage du positionneur, le démontage est **obligatoire** :

1. Dévisser complètement la molette de la tige ;
2. Sortir la tige du corps du positionneur ;
3. Dévisser complètement la molette de la noix ;
4. Sortir la noix du corps du positionneur ;
5. Dévisser la molette du corps
6. Vérifier l'absence de résidus organiques sur le pas de vis du corps
7. Nettoyer chaque pièce séparément ;
8. Remonter le tout sans forcer.

Instruction for assembly and disassembly for cleaning

Humeral cutting guide positioner

Associated reference	Designation	Number of components
B2293311	Humeral cutting guide positioner	3

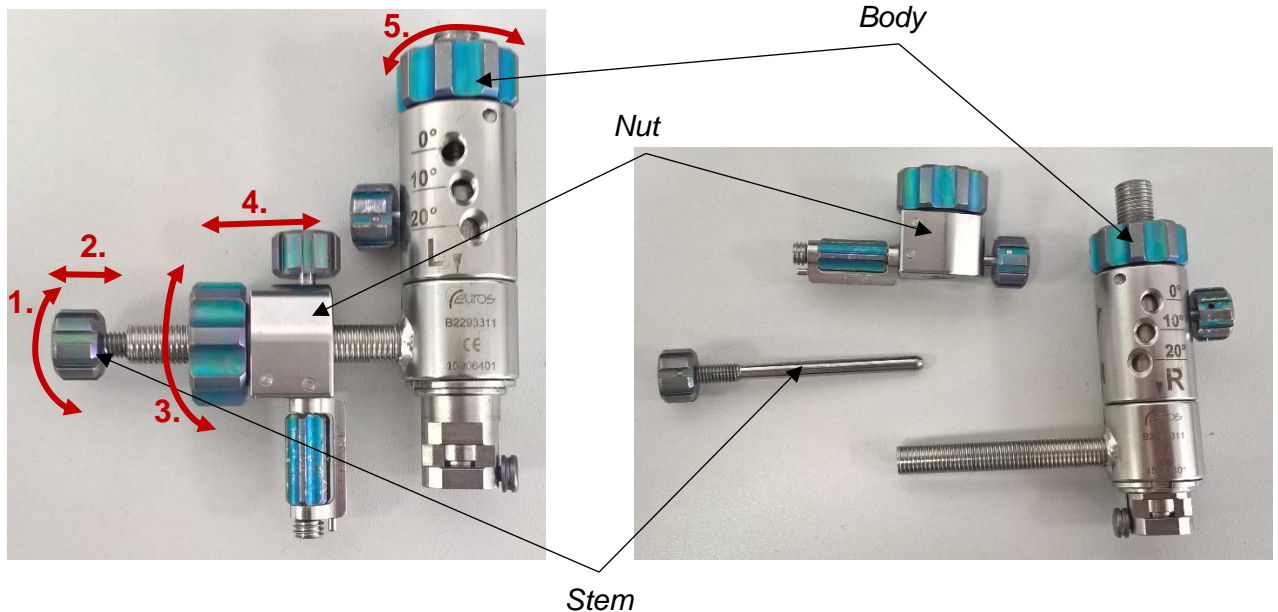

Steps for assembly/disassembly for cleaning

For the cleaning of the condyle holder, the disassembly is **obligatory**:

1. Totally unscrew the wheel from the stem;
2. Take the step out of the body of the positioner;
3. Totally unscrew the wheel from the nut;
4. Take the nut out of the body of the positioner;
5. Unscrew the wheel of the body;
6. Visually check that there are no organic residues on the screw part of the body
7. Clean each part separately;
8. Assemble the set without force.