


## Notice de démontage/remontage pour nettoyage Distracteur Pédiculaire

Références concernées	Désignation	Nombre de composants
B266503001	Distracteur pédiculaire	2

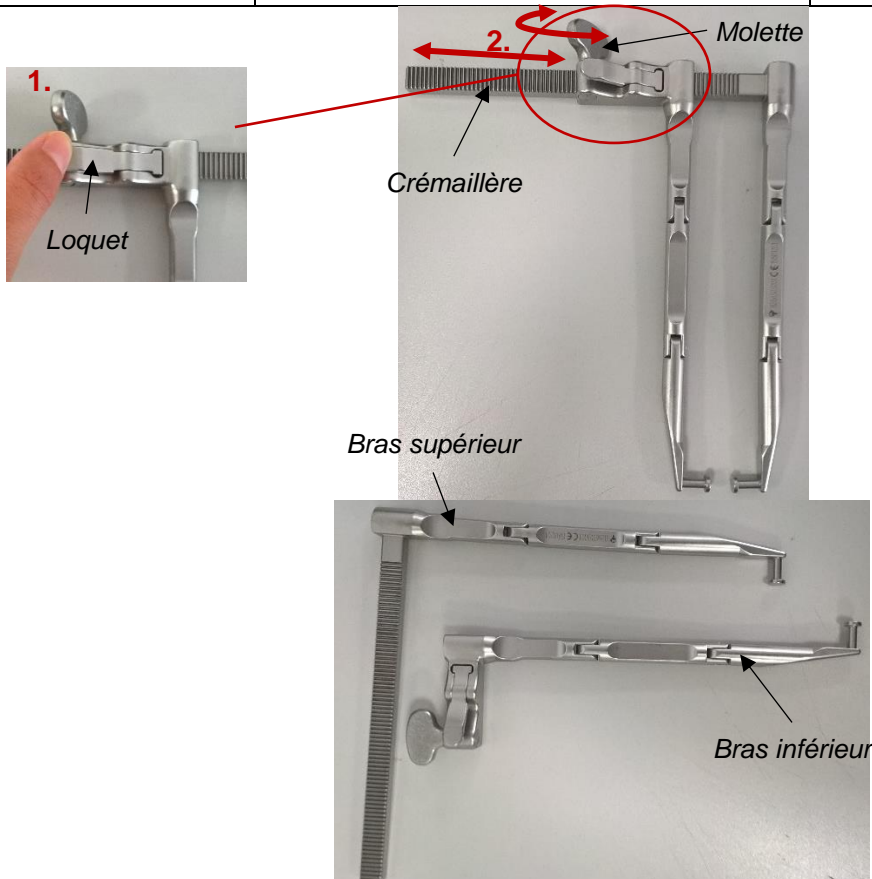

### Etapes de démontage/remontage pour nettoyage




Pour le nettoyage du distracteur, le démontage est **obligatoire**:

1. Débloquer le loquet de verrouillage de position de la crémaillère ;
2. Dévisser complètement la molette et sortir le bras inférieur du corps du distracteur ;
3. Nettoyer chaque pièce séparément ;
4. Remonter le tout sans forcer, à l'aide de la molette ;
5. Bloquer en position fermée avec le loquet.

## Instruction for assembly and disassembly for cleaning

# Pedicle distractor

Associated reference	Designation	Number of components
B266503001	Pedicle distractor	2

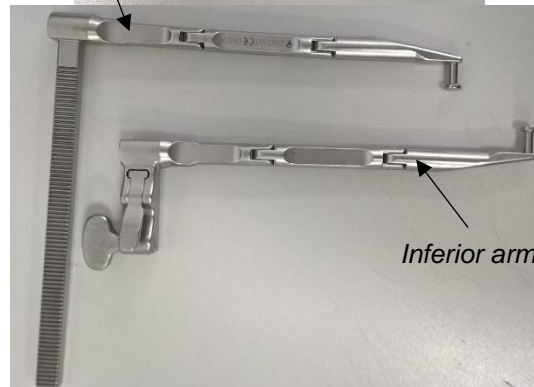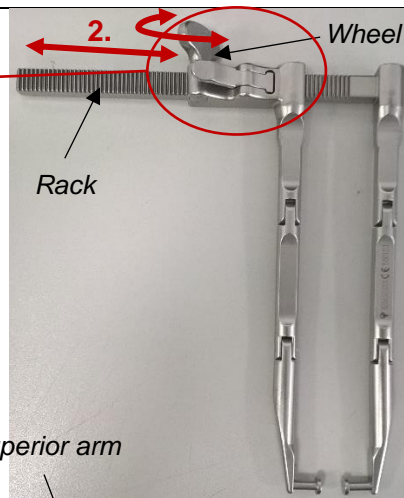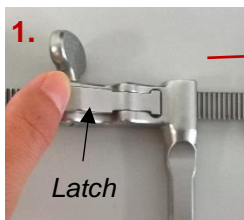

### Steps for assembly/disassembly for cleaning

For the cleaning of the pedicle distractor, the disassembly is **obligatory**:

1. Unlock the latch locking the rack;
2. Totally unscrew the wheel, taking out the inferior arm off the body of the distractor;
3. Clean each part separately;
4. Assemble the set without force helping by the wheel;
5. Lock the latch.